18

REPARATION ENTRETIEN

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MAINTENANCE

DECONTAMINATION, CLEANING AND STERILISATION OF SURGICAL INSTRUMENTS

BASIC RECOMMENDATIONS

- *Handle the surgical instruments with care throughout all the different stages of treatment.*
- *Thoroughly clean new SURGICAL INSTRUMENTS before using them for the first time.*
Two machine washes are advisable.
- *Treat microsurgical instruments separately.*
- *SURGICAL INSTRUMENTS coming into contact with corrosive products must be wiped meticulously.*
- *Respect the manufacturer's instructions for use of the equipment (washing machines, ultrasonic sound machines, sterilizers).*
- *Ensure that the concentration of the decontamination, cleaning and lubricating solutions corresponds correctly to the length of time necessary for it to be effective.*
- *Instruments soiled with organic substances must only be soaked in decontamination solutions and as quickly as possible.*
- *Correctly position the instruments, without overloading, to avoid any obscure areas.*
- *Open out articulated instruments for decontamination and cleaning.*
- *Ban the use of metal brushes and scouring pads.*
- *Rinse thoroughly after decontamination and cleaning.*
- *Carry out the controls for cleanliness and the correct functioning of the SURGICAL INSTRUMENTS.*
- *Sort out and remove the surgical instruments from the lubricants if necessary.*
- *Apply lubricating products to the SURGICAL INSTRUMENTS if appropriate.*
- *Close the rack on the first notch only, for SURGICAL INSTRUMENTS equipped with this mechanism.*
- *Choose packaging adapted to the SURGICAL INSTRUMENTS for sterilisation.*
- *Sterilisation neither replaces decontamination nor cleaning. Good sterilisation can only be carried out on clean items.*